

proKlense

Mouthwash Tablets

DIRECTIONS FOR USE

FEATURES:

proKlense tablets are mouth rinse tablets that rapidly dissolve in water.

INDICATIONS:

For the freshening of the patient's mouth.

For the mechanical removal of post treatment debris.

CONTRA-INDICATIONS:

None.

CONTENTS

1.000 Tablets

PROCEDURE / PRECAUTIONS

(1) APPLICATION:

Dissolve one tablet in half a tumbler of warm water.

SPECIAL NOTICE:

Active Ingredients: Sodium benzoate, Potassium benzoate, Spearmint Oil, Saccharin, Effervescent Base.

Always replace cap tightly immediately after use.

WARNING:

Do not expose patients or users known to be allergic to any of the ingredients of this material. Avoid contact with eyes. In the event of contact, rinse with copious quantities of water and seek medical assistance if symptoms persist.

Do not ingest undiluted tablets.

STORAGE:





Store in a cool dry place (5-25°C).

BATCH CODE:



The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.



The expiry date is shown in year, month format. Do not use the product after this date.

DISPOSAL

Dispose of in compliance with all local and national regulations. Do not reuse empty packaging.



AH1510

1,000 Tablets

This product has been formulated for use in dentistry and is intended for use by dental professionals only.





Advanced Healthcare Ltd., Tonbridge, Kent, TN11 8JU, UK
Tel: +44 1892 870500 Fax: +44 1892 870482 Email: sales@ahl.uk.com

AHL operate a policy of continuing surveillance & monitoring of our products. If you experience any incidents relating to the use of this product, please immediately contact us at the above address stating the batch number shown on the packaging.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a dental professional. AP10215/1

EC REP

REP Advena Ltd. Tower Business Centre, 2nd Flr., Tower Street, Swatar, BKR 4013 Malta,