

AHL

proRinse

Concentrated Mouthrinse

DIRECTIONS FOR USE

FEATURES:

proRinse is a super concentrated mouth rinse to be diluted in water.

INDICATIONS:

For the freshening of the patient's mouth.

For the mechanical removal of post treatment debris.

CONTRA-INDICATIONS:

None.

CONTENTS

100ml Liquid

PROCEDURE / PRECAUTIONS

(1) APPLICATION:

1. Dispense one dose of concentrate into an empty beaker
2. Fill beaker with water
3. Patient should rinse mouth with diluted product and spit it out

SPECIAL NOTICE:

Always replace cap tightly immediately after use.

WARNING:

Do not expose patients or users known to be allergic to any of the ingredients of this material.

Avoid contact with eyes. In the event of contact, rinse with copious quantities of water and seek medical assistance if symptoms persist.

Do not use undiluted.

STORAGE:



Store in a cool dry place (5-25°C).

BATCH CODE:



The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.



The expiry date is shown in year, month format. Do not use the product after this date.

DISPOSAL

Dispose of in compliance with all local and national regulations. Do not reuse empty packaging.



AH1500

100ml Liquid

This product has been formulated for use in dentistry and is intended for use by dental professionals only.



Advanced Healthcare Ltd., Tonbridge, Kent, TN11 8JU, UK

Tel: +44 1892 870500 Fax: +44 1892 870482 Email: sales@ahl.uk.com

AHL operate a policy of continuing surveillance & monitoring of our products. If you experience any incidents relating to the use of this product, please immediately contact us at the above address stating the batch number shown on the packaging.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a dental professional.
2021-05 AP10216/1



Avena Ltd. Tower Business Centre, 2nd Flr., Tower Street, Swatar, BKR 4013 Malta.

