

PROPHYLAXIS PASTE

DIRECTIONS FOR USE

FEATURES:

Contains only natural flavourings and no fluoride. The blends of abrasives used in the various grades are selected to maximise the removal of plaque and tartar without appreciable abrasion of enamel. The thixotropic properties are carefully controlled to ensure easy transfer to cup or brush without undesirable spattering at the rotation speeds used in prophylaxis.

INDICATIONS:

Use for the removal of plaque, tartar and scale from the tooth surface.

CONTRA-INDICATIONS:

None

PROCEDURE / PRECAUTIONS

(1) APPLICATION:

Fill rubber cup / bristle brush with paste and run at low speed over each tooth surface, making sure that the edge of the cup / bristle brush goes below the free margin of the gingiva into the gingival sulcus. Repeat this procedure for each tooth until all exposed surfaces have been treated. Unwaxed dental floss may be used to remove any excess paste through interproximal areas.

SPECIAL NOTICE:

The prophypaste is thixotropic, so, if desired fluidity can be increased by gentle spatulation before use.

Replace lid after use.

For best results, use a prophy type handpiece and rubber cup / bristle brush combination.

STORAGE:



Store in a cool dry place (5-25°C).

WARNING:

Do not expose patients or users known to be allergic to any of the ingredients of this material. Avoid contact with eyes, and skin. In the event of contact, rinse with copious quantities of water and seek medical assistance if symptoms persist.

BATCH CODE:



The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.



The expiry date is shown in year, month format. Do not use the product after this date.

DISPOSAL

Dispose of in compliance with all local and national regulations. Do not empty into drains. Do not reuse empty packaging.

REF			
014X1	Medium Grain Orange Oil Free	300g	
019X1	Coarse Grain Mint Oil Free	300g	
032X1	Coarse Grain Mint Regular	250g	
033X1	Coarse Grain Orange Regular	250g	
034X1	Medium Grain Orange Regular	250g	
036X1	Medium Grain Mint Oil Free	300g	
039X1	Coarse Grain Orange Oil Free	300g	
040X2	Medium Grain Mint Regular	250g	
041X2	Medium Grain Thymol Regular	300g	
114X2	Medium Grain Citrus Regular	300g	
303X3	Medium Grain Bubblegum Regular	250g	
304X3	Medium Grain Strawberry Regular	250g	
408X2	Mint Regular	200g	

This product has been formulated for use in dentistry and is intended for use by dental professionals only.



Advanced Healthcare Ltd., Tonbridge, Kent, TN11 8JU, UK

Tel: +44 1892 870500 Fax: +44 1892 870482 Email: sales@ahl.uk.com

AHL operate a policy of continuing surveillance & monitoring of our products. If you experience any incidents relating to the use of this product, please immediately contact us at the above address stating the batch number shown on the packaging.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a dental professional.

2021-08

AP8336/2

EC REP Advena Ltd. Tower Business Centre, 2nd Flr., Tower Street, Swatar, BKR 4013 Malta.

