

# AHL

## AHfil TC

Tooth Conditioner

### DIRECTIONS FOR USE

#### FEATURES:

AHfil TC is a solution of polyacrylic acid designed for removing the salivary pellicle on uncut or contaminated tooth surfaces thus enhancing the adhesion of glass ionomer restoratives.

#### INDICATIONS:

Removal of the salivary pellicle on uncut or contaminated tooth surfaces

#### CONTRA-INDICATIONS:

Keep away from pulp cavity.

Line deep cavities with a calcium hydroxide liner before using the Toothcleanser.

#### CONTENTS

Liquid 10ml, instructions for use

#### PROCEDURE / PRECAUTIONS

##### (1) APPLICATION:

Apply to surface using a pledget of cotton wool for a maximum of 30 seconds. If freshly cut dentine or enamel is contaminated with saliva, apply for 10 seconds only. Wash with water and dry with oil-free air. Do not desiccate the tooth. Fill immediately with glass ionomer restorative.

##### SPECIAL NOTICE:

Always replace cap tightly immediately after use.

##### WARNING:

Contains 25% polyacrylic acid - avoid contact with eyes. In case of contact, wash thoroughly with water and obtain medical advice. Do not expose patients or users known to be allergic to this type of material.

##### STORAGE:



Store in a cool dry place (5-25°C).

##### BATCH CODE:



The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.



The expiry date is shown in year, month format. Do not use the product after this date.

##### DISPOSAL

Dispose of in compliance with all local and national regulations. Do not reuse empty packaging.



AH0250 10ml Liquid

This product has been formulated for use in dentistry and is intended for use by dental professionals only.



Advanced Healthcare Ltd., Tonbridge, Kent, TN11 8JU, UK

Tel: +44 1892 870500 Fax: +44 1892 870482 Email: [sales@ahl.uk.com](mailto:sales@ahl.uk.com)

AHL operate a policy of continuing surveillance & monitoring of our products. If you experience any incidents relating to the use of this product, please immediately contact us at the above address stating the batch number shown on the packaging.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a dental professional.  
2021-05 AP9157/2



Advvena Ltd. Tower Business Centre, 2nd Flr., Tower Street, Swatar, BKR 4013 Malta.

